Ureter-Dauerschienen HEPARIUS®

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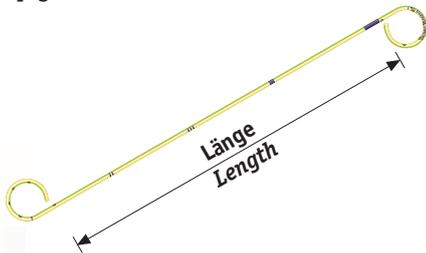
To facilitate the choice our data regarding the length relate to the ureteral part of the Ureteral Stents without pigtails.

Charrière	Millimeter
4,8	1,6
6,0	2,0
7,0	2,3

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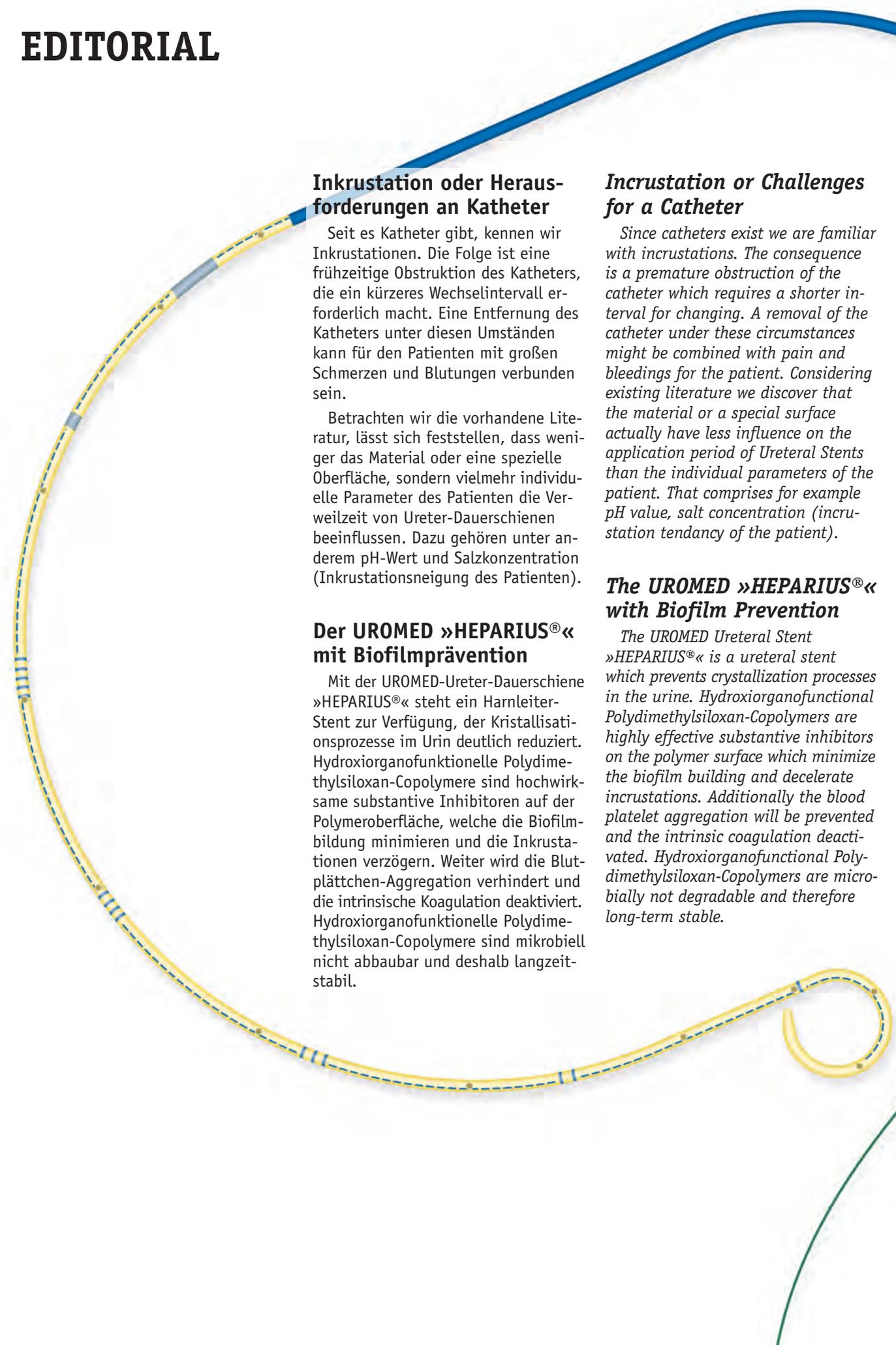
Incrustation or Challenges for a Catheter

Since catheters exist we are familiar with incrustations. The consequence is a premature obstruction of the catheter which requires a shorter interval for changing. A removal of the catheter under these circumstances might be combined with pain and bleedings for the patient. Considering existing literature we discover that the material or a special surface actually have less influence on the application period of Ureteral Stents than the individual parameters of the patient. That comprises for example pH value, salt concentration (incrustation tendency of the patient).


The UROMED »HEPARIUS®« with Biofilm Prevention

The UROMED Ureteral Stent »HEPARIUS®« is a ureteral stent which prevents crystallization processes in the urine. Hydroxiorganofunctional Polydimethylsiloxan-Copolymers are highly effective substantive inhibitors on the polymer surface which minimize the biofilm building and decelerate incrustations. Additionally the blood platelet aggregation will be prevented and the intrinsic coagulation deactivated. Hydroxiorganofunctional Polydimethylsiloxan-Copolymers are microbially not degradable and therefore long-term stable.

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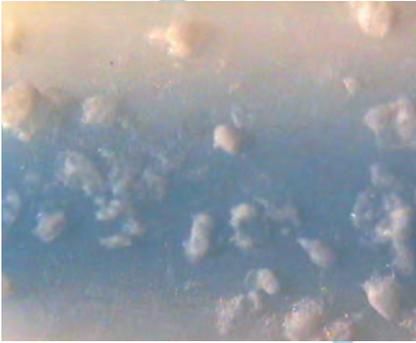Massive deposits at a conventional Ureteral Stent made of polyurethane after 7 days of rinsing experiment.

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No deposits at the UROMED »HEPARIUS®« after 7 days of rinsing experiment.

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The Efficacy of the UROMED »HEPARIUS®«

The UROMED stent »HEPARIUS®« is characterized by constantly good drainage capacity, even when applied for a period of up to 12 months. Therefore a reduced number of exchange interventions and a high wearing comfort can be achieved for the patients.

Comparing stents with additives, based on Polydimethylsiloxan-Copolymers, and polyurethane stents without additives, remarkable differences concerning crystal deposits are found. Hence an important extension of indwelling time of the UROMED »HEPARIUS®« in relation to standard stents can be achieved.

The Ureteral Stents

The admixture of an additive to the polyurethane which essentially prolongs the indwelling time puts specific demands on the product basis, the Ureteral Stent. Therefore we paid great importance to the UROMED »HEPARIUS®« regarding tensile strength, form stability, reliable memory features and flexibility. Our connection allows to the user a safe and fast application.

The material strength in combination with the small-sized, tangentially moved atraumatic eyes prevents from compression during the application.

Due to the Biofilm Prevention and the high material strength the new »HEPARIUS®« without perforation in the ureteral part is also perfectly suitable for tumor patients.

Product Features:

- incrustation-reducing additive
- reduced bacterial adherence
- lower strain on the patient due to reduced number of surgeries
- excellent for long-term application
- polyurethane

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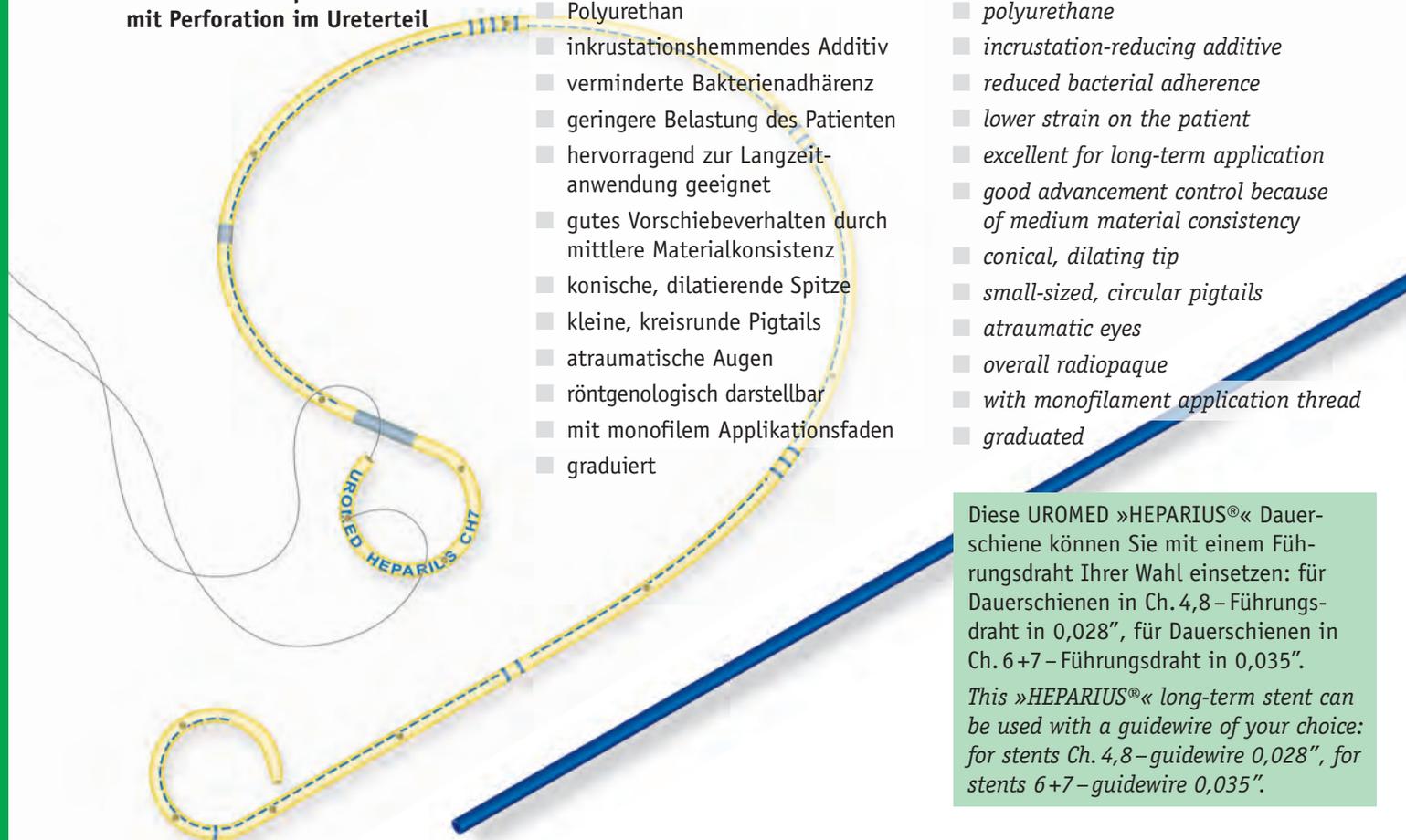
Catheter for the internal splinting of the ureter

- polyurethane
- incrustation-reducing additive
- reduced bacterial adherence
- lower strain on the patient
- excellent for long-term application
- good advancement control because
of medium material consistency
- conical, dilating tip
- small-sized, circular pigtails
- atraumatic eyes
- overall radiopaque
- with monofilament application thread
- graduated

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*ureteral stent, double pigtail
central open tip, with perforation
in the ureteral part*

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Right after positioning of the ureteral stent the application thread has to be removed.

REF REF	Dauerschienen Stent	PZN PZN
4145-24	Ch. 4,8/24 cm	11370579
4145-26	Ch. 4,8/26 cm	11370585
4145-28	Ch. 4,8/28 cm	11370591
4145-30	Ch. 4,8/30 cm	11370616
4146-24	Ch. 6,0/24 cm	11370622
4146-26	Ch. 6,0/26 cm	11370639
4146-28	Ch. 6,0/28 cm	11370645
4146-30	Ch. 6,0/30 cm	11370651

REF REF	Dauerschienen Stent	PZN PZN
4147-24	Ch. 7,0/24 cm	11370668
4147-26	Ch. 7,0/26 cm	11370674
4147-28	Ch. 7,0/28 cm	11370705
4147-30	Ch. 7,0/30 cm	11370711

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Ureter-Dauerschiene
beidseitiger Pigtail
*ureteral stent, double pigtail
central open tip, without
perforation in the ureteral part*

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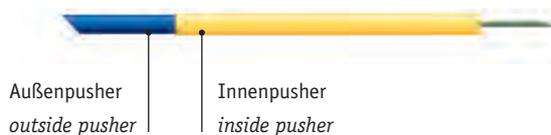
The »HEPARIUS®« Ureteral Stents are available with two different connections

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SEPARATION OF THE CONNECTION:

Draw back the yellow inside pusher and hold on to the blue outside pusher at the same time to disconnect the stent.

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FIXING THE GUIDEWIRE

The ureteral stent and the pusher system are tightly connected. For a fixation of the guidewire please close the rotation thread at the outer end of the pusher system.

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
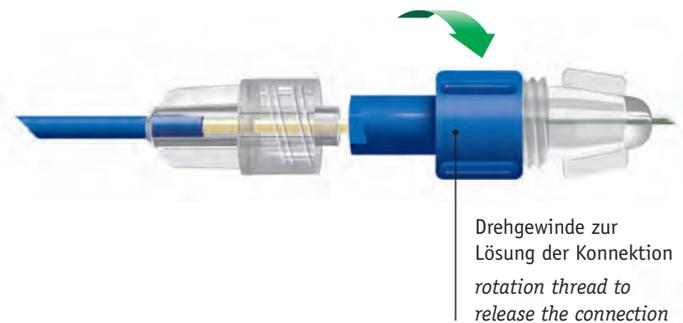
DISCONNECTION BETWEEN STENT AND PUSHER SYSTEM

Separation of the connection by unscrewing (1.) and drawing back (2.) the blue connector which is linked to the yellow inside pusher.

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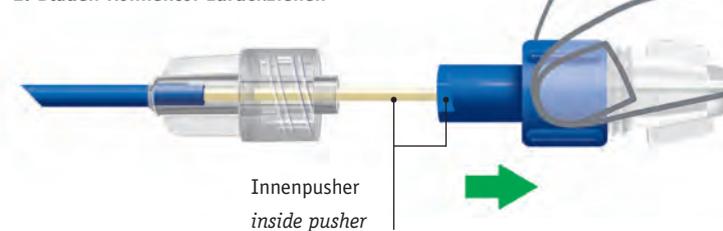*Catheter for the internal splinting
of the ureter*

- *polyurethane*
- *incrustation-reducing additive*
- *reduced bacterial adherence*
- *lower strain on the patient*
- *excellent for long-term application*
- *good advancement control because of medium material consistency*
- *conical, dilating tip*
- *small-sized, circular pigtails*
- *atraumatic eyes*
- *pre-assembled set for easy handling*
- *overall radiopaque*
- *graduated*

Ureter-Dauerschleie
beidseitiger Pigtail
*ureteral stent, double pigtail
cylindrical tip, with perforation
in the ureteral part*

Ch. 6, Ch. 7
Pushersystem
Länge 45 cm
*Ch. 6, Ch. 7
pusher system
length 45 cm*

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Catheter for the internal splinting
of the ureter

- polyurethane
- incrustation-reducing additive
- reduced bacterial adherence
- lower strain on the patient
- excellent for long-term application
- good advancement control because of medium material consistency
- conical, dilating tip
- small-sized, circular pigtails
- atraumatic eyes
- pre-assembled set for easy handling
- overall radiopaque
- graduated

Ureter-Dauerschleife
beidseitiger Pigtail
*ureteral stent, double pigtail
cylindrical tip, with perforation
in the ureteral part*

Ch. 6, Ch. 7
Pushersystem
Länge 45 cm
*Ch. 6, Ch. 7
pusher system
length 45 cm*

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*All indications correspond to our know-
ledge at the date of press.
Due to technical progress all products are
subject to change.*

09/2015

© UROMED Kurt Drews KG